




 We recommend using high viscosity silicone impression material. If a mixing-tip style material must be used, we require to use a napkin to press on the material after it's expressed into the ear.

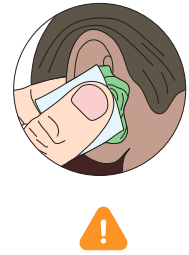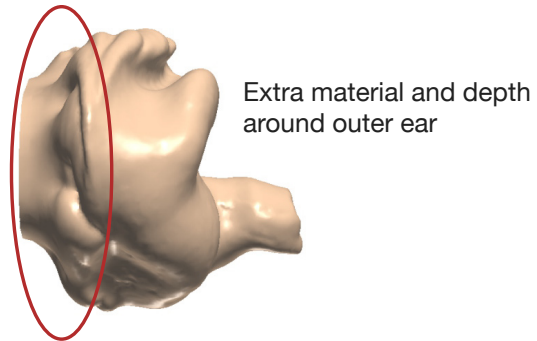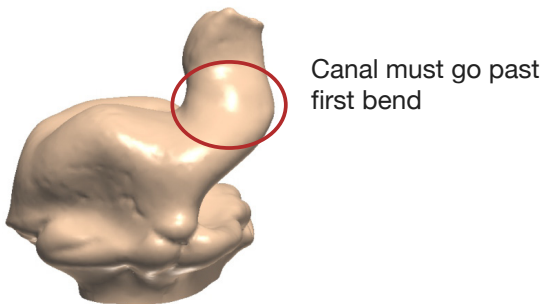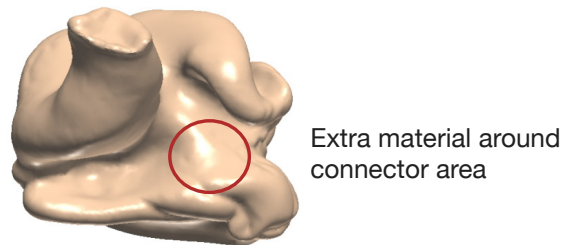

For best fit, make open-mouth impressions by using a 1/2" or 1" bite block. Custom In-Ear Monitors require a complete mold of the ear without large voids or bubbles. Please see details:

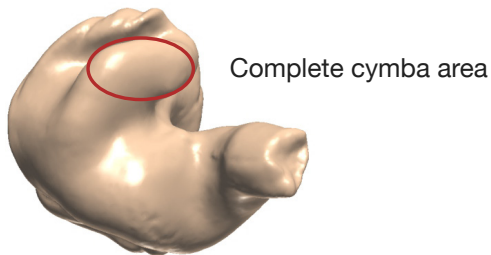

- 1 Take at least 3 photos (from different angles) of your impressions and email them to [info@bellosaudio.com](mailto:info@bellosaudio.com) for approval. In case your impressions need to be remade, this step will save you lost time and extra shipping fees.
- 2 Once your ear impressions are approved over email, it's time to mail them to Bellos Audio. Make sure to include your name, email address, and phone number with your shipment. Mail your impressions to:

**Bellos Audio**  
3511 Gallatin Pike, Ste 209  
Nashville, TN 37216

**STL**

We accept digitally scanned impressions in STL file format. Email them to [info@bellosaudio.com](mailto:info@bellosaudio.com) and include photos of the physical impressions as a reference (if you have them).