





We recommend using high viscosity silicone impression material. If a mixing-tip style material must be used, we require to use a napkin to press on the material after it's expressed into the ear.




Due to the In-Air Canals technology, we only require the impression material to go past the 1st bend in canal. A bite block is **not required** but can be used if audiologist deems necessary. Custom In-Ear Monitors require a complete mold of the ear without large voids or bubbles. Please see details:

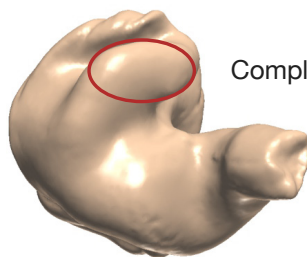


Complete cymba area

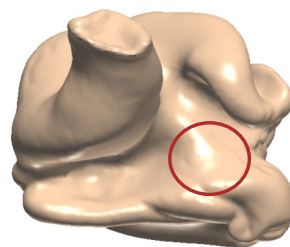

Extra material around connector area

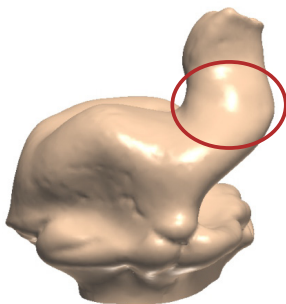

Canal must go at least past first bend

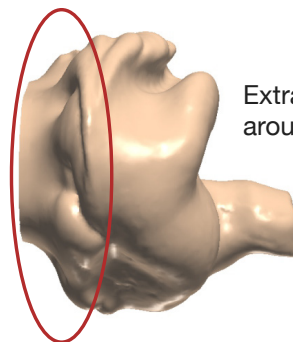

Extra material and depth around outer ear

- 1 Take at least 3 photos (from different angles) of your impressions and email them to info@bellosaudio.com for approval. In case your impressions need to be remade, this step will save you lost time and extra shipping fees.
- 2 Once your ear impressions are approved over email, it's time to mail them to Bellos Audio. Make sure to include your name, email address, and phone number with your shipment. Mail your impressions to:

Bellos Audio
12901 SE 97th Ave, Ste 360
Clackamas, OR 97015

STL

We accept digitally scanned impressions in STL file format. Email them to info@bellosaudio.com and include photos of the physical impressions as a reference (if you have them).